

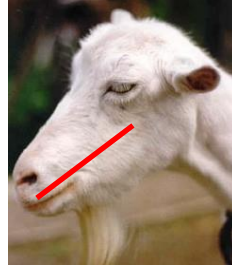
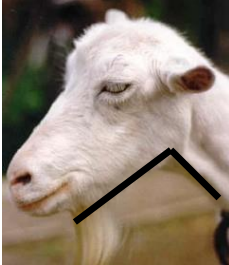


User manual for inserting the bolus

- The bolus can only be inserted in animals of at least 25 kg. This means it cannot be used for small breeds.
- Before you start, use a reader to check that there is not already a bolus in the animal.
- Secure the animal and hold the head in its natural position (90°).

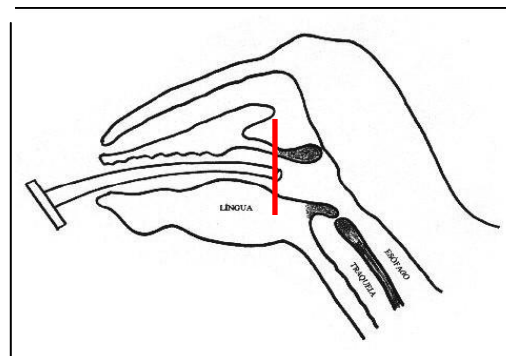

- Check the length of the bolus applicator in relation to the size of the animal's head.
- Open the animal's mouth by gently pressing the palate behind the incisors using your fingertips.
- Gently slide the bolus applicator in the mouth until it reaches the end of the tongue. **Never force or coercively push the bolus applicator in!!**
- Press the handle to insert the bolus. You will see the bolus slide down on the left-hand side of the neck.
- If the bolus stays in the mouth, because the animal hasn't swallowed it, wait calmly until the animal spits it out.
- If required, the bolus can be read after insertion, using a hand reader on the left-hand side of the animal.
- Mark the animal, to avoid a second bolus being inserted

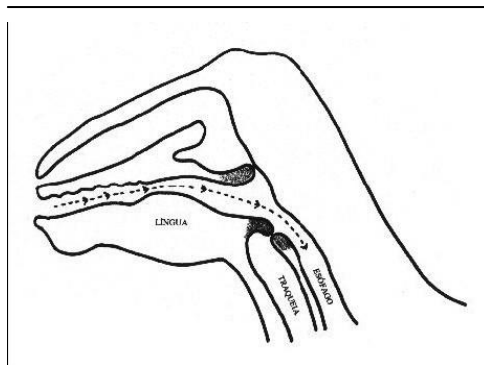

For attaching the ear tag, see the relevant user manual.

The ELDA SG ear tag can, for example, be inserted using the application pliers as shown below and the correct point.

